

 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-40 00
F +49 (0) 36628 66-44 99
E info@bauerfeind.com
BAUERFEIND.COM


RhizoLoc®

Daumenorthese
Thumb orthosis
Orthèse de pouce
Ortesis de pulgar
Tutore stabilizzante per Pollice


 **MADE
IN
GERMANY**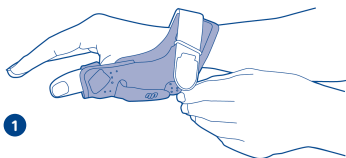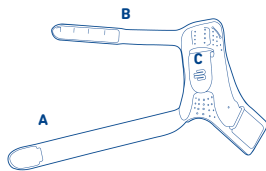


1

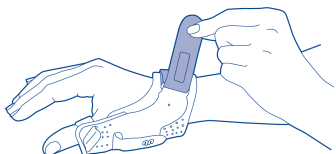2

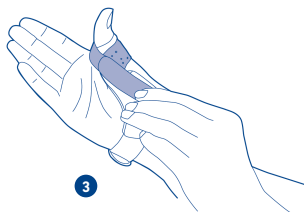3

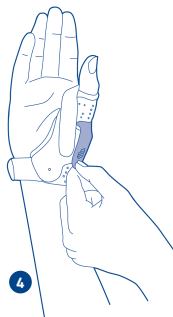4

(de)	deutsch.....	3	(hr)	hrvatski	48
(en)	english.....	6	(sr)	srpski.....	50
(fr)	français	9	(sl)	slovenski.....	53
(nl)	nederlands.....	12	(ro)	romanian.....	56
(it)	italiano.....	15	(el)	ελληνικά.....	59
(es)	español.....	18	(tr)	türkçe.....	62
(pt)	português.....	21	(ru)	русский.....	65
(sv)	svenska.....	24	(lv)	latviešu.....	69
(no)	norsk.....	27	(li)	lietuvių.....	72
(fi)	suomi.....	30	(he)	תלרברר.....	76
(da)	dansk.....	33	(ja)	日本語.....	78
(pl)	polski.....	36	(ko)	한국어.....	81
(cs)	česky.....	39	(ar)	عربي.....	84
(sk)	slovensky.....	42	(zh)	中文.....	87
(hu)	magyar.....	44			

(de) deutsch

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Polyurethan (PUR), Polyamid (PA), Aluminium (Al), Polyester, Polyester (PES), Polyoxymethylen (POM)

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 english

Dear Customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

RhizoLoc is a medical device. It's an orthosis used for the stabilization of the metacarpophalangeal and carpometacarpal joints of the thumb.

Indications

- Post-operative and post-traumatic irritation
- Collateral ligament injuries, e.g. skier's thumb
- Osteoarthritis of the carpometacarpal joint of the thumb (mild)
- Sprains / contusions

Risks of using this product

Caution*
Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

RhizoLoc must only be worn in accordance with the specifications contained in these instructions and used for the areas of application (places of use) listed. To guarantee the best possible fit of the RhizoLoc, it is essential to check whether the product is adequately shaped and whether it needs to be anatomically reshaped by a specialist². Only a trained specialist may carry out

the (initial) fitting of the orthosis and provide instruction.

The sewn-in label of the RhizoLoc, which contains information on the product name, size, manufacturer, washing instructions and CE certification, is located on Rhizobic outside of the wrist strap.

Use in combination with other products, e.g. as part of compression treatment (arm stockings), must be discussed beforehand with your physician. Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay. No product liability is accepted in the event of improper use.

Avoid the use of ointments, lotions or any substances including grease or acids. Do not make any alterations to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed.

Do not put the RhizoLoc on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the straps of the RhizoLoc and, if necessary, check the size of your RhizoLoc.

Contraindications

Hypersensitivity reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and worn after consultation with your physician:

- skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised

scars with swelling, redness and excessive heat build-up

- impaired sensation and circulatory disorders of the arm / hand (e.g. diabetes mellitus)
- impaired lymph drainage, including soft tissue swellings of uncertain origin located away from the fitted orthosis

Application instructions

Putting on the RhizoLoc

A – wrist strap, B – thumb strap, C – thumb tab

- 1 Open the two Velcro straps on the RhizoLoc. Loosen the straps without pulling them out of the guide. **2** Put the RhizoLoc on the affected thumb. To do this, guide your hand through the bottom wrist strap and your thumb through the top thumb strap.
- 3 Close the wrist strap. Then close the thumb strap by closing the Velcro at the side next to the thumb tab.
- 4 Following consultation with your physician, you can use the thumb tab to adjust the level of stabilization of the thumb basal joint depending on the phase of your treatment.

Taking off the RhizoLoc

Open the thumb and wrist straps. Guide your hand downward out of the RhizoLoc.

Cleaning instructions

Please note the following:

- Remove the Velcro straps if possible, or fasten them securely in position.
- Air dry the product. Do not dry it in a tumble-dryer because this can damage the product.
- Please only wash the orthosis¹ by hand at 30 °C using a mild detergent. All components can stay on the orthosis¹ during washing. Leave the orthosis¹ to air dry.

- Regular care guarantees optimum effectiveness of the product. Also observe the instructions on the sewn-in label on the upper edge of your product.
- Never expose the product to direct heat as this may impair its effectiveness. If you have any complaints, please contact your retailer. Only clean goods are handled.

Part of the body this product is used for

In accordance with the indications (thumb). See intended use.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The RhizoLoc is supplied in an anatomically contoured state. If necessary, the RhizoLoc can be contoured further. The orthosis is available in right- and left-handed designs.

Technical specifications / parameters

The RhizoLoc is a orthosis for the thumb. There are no spare parts available.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited

or excluded if the instructions on how to handle and care for the RhizoLoc have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyurethane (PUR), Polyamide (PA), Aluminium (Al), Polyether, Polyester (PES), Polyoxymethylene (POM)

 – Medical Device
 – DataMatrix Unique Device Identifier UDI

Version: 2022-04

Specialists²

General notes

- Check the optimal and individual fit of the RhizoLoc after the first fitting. Make corrections if necessary.
- Practice the correct procedure for putting on the orthosis with the patient.
- The product must not be changed in any improper way. Failure to comply with this requirement may adversely affect product performance,

thereby voiding any product liability.

- The RhizoLoc is intended for the treatment of one patient only.

The aluminum stays are anatomically contoured, but they may still need to be individually adjusted to suit the patient's anatomy. To do this, apply a little force to contour the aluminum stays into the anatomically correct form.

The thumb tab to limit the thumb basal joint must be firmly strapped in and under tension when treatment begins and must contain no folds. It is the physician's responsibility to adjust the level of restriction of movement of the thumb.

- * Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product)

1 Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

2 A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.