EpiTrain®

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INSTRUCTIONS FOR USE

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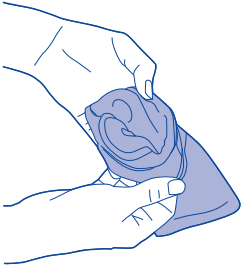
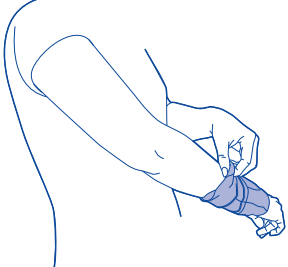
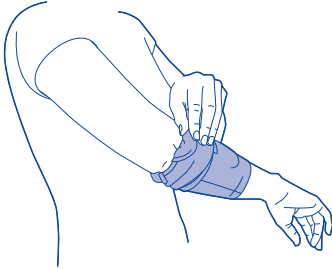
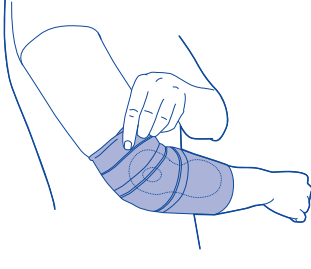
RELIEVES AND MOBILIZES

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1		2	
3		4	
DE deutsch	03	SR srpski	25
EN english	04	SL slovenski	27
FR français	05	RO romanian	28
NL niederlands	07	EL ελληνικά	29
IT italiano	08	TR türkçe	31
ES español	10	RU русский	32
PT português	11	ET eesti	34
SV svenska	13	LV latviešu	35
NO norsk	14	LT lietuvių	36
FI suomi	16	UK українська	38
DA dansk	17	KK қазақ	39
PL polski	18	JA 日本語	41
CS česky	20	HE עברית	43
SK slovensky	21	ZH 中文	44
HU magyar	23	AR عربي	45
HR hrvatski	24		

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 **english**

Dear patient, thank you for choosing a Bauerfeind product.

We wish you a speedy recovery.
Please read and observe these instructions for use closely and in their entirety. If there is something you do not understand or if you have any questions, do not use the product until you have consulted your physician or professional¹ again. Keep these instructions for use in case you want to look something up later.

Intended purpose
EpiTrain is a medical device. The elbow support is a support for the relief and stabilization of the elbow joint.

- Indications**
- Post-traumatic or post-operative irritation, e.g. epicondylitis
 - Osteoarthritis / arthritis (joint wear / joint inflammation)
 - Tendinopathy (degenerative tendon condition)
 - Tendomyopathy (muscle-tendon condition)
 - Prevention / recurrence prevention

- Contraindications**
For the following clinical pictures, usage of the product is only indicated upon consultation with your physician:
- Skin disorders/lesions on the part of the body to be treated, especially if inflammatory manifestations as well as raised scars with swelling, redness and overheating are present
 - Abnormal sensation in the arm
 - Impaired circulation in the arm
 - Impaired lymphatic drainage – including unspecified soft tissue swelling located in distal direction of the fitted aid

Ask your physician whether your conditions fall into these categories.

Notes before first usage
If you wish to use your Bauerfeind product for acute symptoms, injuries or healed previous conditions, you urgently need to obtain professional medical advice and observe it before using the product. Restrictions may apply to you which you must take into account and which the professional¹ will clarify.

In case of identified or suspected problems with blood pressure and intense physical exercise, prolonged use of the product can have additional effects on your blood pressure.

After you have been prescribed/you have purchased the product, only use it according to the indication and under observance of any additional instructions given by your physician/professional¹. If you wish to use the product in combination with medication, other medical devices or other treatments, consult your physician/professional¹ beforehand.

If any additional conditions or injuries occur during the period of application of the product, also consult your physician immediately.

- Risks of application** 
- Do not arbitrarily modify the product as this may cause it to not help as expected and/or worsen your health.
 - The product must not be applied too firmly as this may cause local pressure symptoms or, in rare cases, constrict underlying blood vessels or nerves.
 - If you notice any changes or an increase in symptoms related to the application of the product (such as tissue congestion, skin irritation, swelling, etc.), pause the application and consult your physician/professional¹.
 - If you completely doff the product, treatment is insufficient and additional damage to the part of the body affected may occur during

- additional strain. Be especially careful when this is the case.
- Additionally, restrictions due to the indication may apply which the professional¹ will explain to you. In particular, driving vehicles, other means of transportation or machines may only be possible under restrictions. The same applies to the use of ladders, tools and sports equipment. When in doubt, we recommend to suspend these activities.
 - Persons requiring supervision due to their age or mental/physical impairment must be supervised during every application of the product.
 - Avoid contact of the product with fatty and acidic agents, ointments, lotions, bases, sand or salt water.
 - If hypersensitivity has been observed with you, consult your physician before the first application of the product. If hypersensitivity/allergic reaction occurs in relation with the application of the product, pause the application and consult a physician immediately.
 - Never expose your product to direct sunlight, heat or cold for longer periods of time.

- Application instructions**
- Observe the instructions for donning and doffing of the product as well as any additional instructions for use given by your physician/professional¹ under any circumstances.
 - Since the product unfolds its effects especially during physical activity, doff it during longer rest phases.
 - Check the product for integrity before every use. Only intact products may be used.
 - Make sure to have your physician/professional¹ show you how to don/doff the product and explain to you how it works until no doubts remain. If the affected part of the body changes in diameter, the product may have to be readjusted/newly prescribed.

Donning of the product
For easy donning, fold the top part of the support outward so that the cutouts for the internal massage pads become visible . Then grab the support near the pad pocket  and pull it over the elbow. Make sure that the outer bony protrusion of the elbow (epicondyle) is positioned in the cut out of the pad, just the same as the inner bony protrusion of the elbow . Then fold the upper edge of the support back into the original position . Doffing of the product
For easy doffing, grab the support at the bottom edge near the pad pocket and pull it toward your hand.

Cleaning instructions
Observe the supplementary instructions on the sewn-in label/sticker on your product. Wash the product at 30 °C/86 °F. Regular care ensures the ideal effectiveness of the product.

Area of use
Elbow joint
Notes on reuse
The product is intended for the individual treatment of one patient only.

Warranty
The legal provisions of the country in which the product was purchased apply. If you suspect a warranty claim, please first contact the person from whom you purchased the product directly. The product must be cleaned before making a warranty claim. If the instructions for handling and caring for the product have not been

observed, the warranty may also be affected or excluded.

- The warranty does not cover:
- Off-label use
 - Failure to observe the instructions of the physician/professional¹
 - Arbitrary product modifications

Disclaimer
In general, do not self-diagnose, self-treat or self-medicate unless you are a professional¹. Before first using our medical device, actively consult a physician or professional¹ since this is the only way to assess the individual effect of the product on your body and determine any possible risks of application/side effects. Please observe the advice of this professional¹ as well as all notes and instructions in this document or its online version (including texts, images, graphics, etc.), including any excerpts.

Obligation to report
Due to regional legal regulations, you are obliged to report any serious incidents involving the use of this medical device to both the manufacturer and the competent authority for medical devices (in Germany: Bundesinstitut für Arzneimittel und Medizinprodukte - Federal Institute for Drugs and Medical Devices).

Disposal
Please dispose of the product in accordance with local regulations.

Material content
Styrene-ethylene/butylene-styrene block copolymer (TPS-SEBS), Polyamid (PA), Elastane (EL), Cotton (CO), Polyurethane (PUR), Polyester (PES)

 – Medical Device
 – DataMatrix identifier as UDI

Date of information: 2026-07

¹ A professional is any person who is authorized to adjust supports and orthoses and instruct others in their use in accordance with the applicable state and institutional regulations and other guidelines. As the manufacturer, we strongly recommend that this person disposes of orthopedic technical training or comparable acquired skills, as well as orthopedic technical infrastructure.

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BAUERFEIND AG