**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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 english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

## Intended purpose

GenuTrain A3 is a medical device. It is a support for activation and stabilization in

the case of complex knee problems such as osteoarthritis.

 **GenuTrain A3 must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.**

## Indications

- Osteoarthritis of the knee and arthritis
- Chronic, post-traumatic and post-operative irritation
- Tendinopathy, ligament insertion degeneration and tendomyopathy
- Knee pain in cases of functional instability due to muscular imbalance
- Recurring joint effusion
- Prevention / relapse prevention
- Patellar lateralization

## Risks of using this product

### Important Information

**Please closely observe the specifications in these instructions for use and the instructions given by the specialist.**

The benefits of this product are felt particularly during physical activity.

- Please remove the support during longer periods of rest.
- Once you have been prescribed the GenuTrain A3, only use it according to your indication and any additional instructions given by a medical specialist\*. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

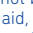
## Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Varicose veins.
3. Impaired sensation in the leg due to neurological abnormalities
4. Impaired circulation in the leg (e.g. diabetes mellitus)
5. Impaired lymphatic drainage and soft tissue swelling of unknown origin

## Application instructions

### Fitting GenuTrain A3

1. Firmly grip the top of the support with both hands (one either side of the lateral spines)  and pull evenly over the knee joint until the knee cap is free from pressure and is surrounded by the silicone insert.  – 
2. Adjust as required until a snug fit is achieved.
3. The bandage must not be put on using the wrong donning aid, as this can destroy the silicon coating.

### Removing GenuTrain A3

To remove the support, pull down at the bottom of the support near the stays. 

## Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product. If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

## Notes on reuse

This product is intended to be fitted to one patient only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuTrain A3 have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content:

Silicone (SI), polyamid (PA), spring steel wire grade A, galvanized elastodiene (ED), spandex (EL), viskose (CV), cotton (CO), polyurethane (PUR), polyester (PES)

 – Medical Device  
 – DataMatrix Unique Device Identifier UDI

Version: 2022-03

\*A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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