Stomacare





Protector**Max**

Basko

Health**care**

basko.com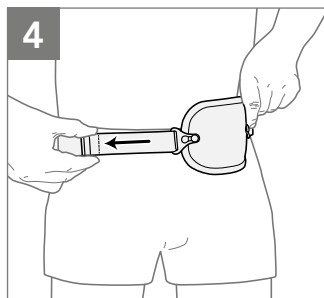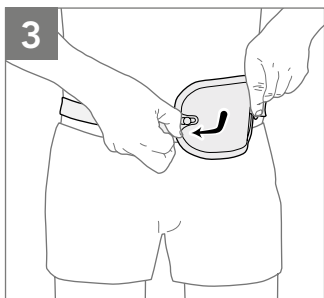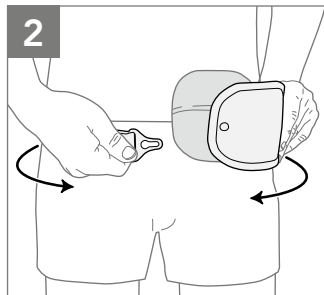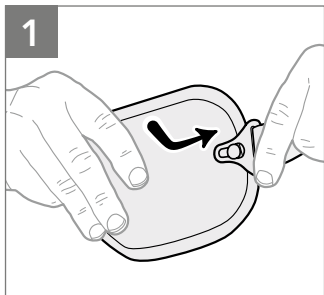

DE	-	Gebrauchsanleitung.....	6
EN	-	Instructions for use.....	8
FR	-	Mode d'emploi.....	10
SE	-	Användarinstruktioner.....	12
NO	-	Bruksanvisning.....	14
DK	-	Brugervejledning.....	16
FI	-	Käyttöohje.....	18
IT	-	Manuale di istruzione.....	20
ES	-	Manual de instrucciones.....	22
PT	-	Manual de instruções.....	24
PL	-	Instrukcję obsługi.....	26

Stomacare ProtectorMax

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Stomacare ProtectorMax

Please read these instructions carefully before using the product. In particular, make sure that you comply with the safety regulations as mentioned.

Instructions for use

Your family, friends, fellow Ostomates or perhaps your wound nurse have advised you to wear a ProtectorMax. For optimal care, the user should read these instructions carefully before using the device. This product is a medical device and is intended for use by one person only.

Intended use of the product

The ProtectorMax is used to protect the stoma and offers the physically active Ostomate additional protection against pressure and undesired force effects on the stoma, e.g. during exercise or pressure on the stoma caused by the seat belt. The ProtectorMax offers Ostomates safety and protection in everyday life.

Packaging content

First check that the contents of the packaging are complete and undamaged. If this is not the case, please contact the person from whom you purchased the product directly. The packaging contains the following parts:

- ProtectorMax
- Elastic strap with adjustable sliding and fastening buckles
- Instruction manual
- Protective box

Applying the ProtectorMax

In order to wear the ProtectorMax correctly, you should preferably put it on lying down when applying for the first time. The strap is tightened enough, in a way a hand can still be placed in between. The ProtectorMax then sits tight enough without migrating, but still wears comfortably.

Pic. 1

After checking the contents of the package, hook one of the fastening buckles at the end of the elastic strap onto the ProtectorMax.

Pic. 2

Now bring the strap back to the front along your backside and position the ProtectorMax at the level of your stoma-opening.

Pic. 3

Now hook the other buckle onto the other side of the ProtectorMax.

Pic. 4

Use one hand to hold the ProtectorMax around your stoma and in front of the stoma bag, while with the other hand you adjust the strap with the adjustable slider to the desired tension.

Additional product information/Application

The ProtectorMax is not intended for use during water sports activities. However, in case of contact with chlorine or saltwater, it is advisable to clean it thoroughly afterwards.

Existing user risks

No user risks are known. Please check your skin condition on a daily base. If the first signs of skin irritation, redness or swelling appear, please remove the ProtectorMax immediately and contact your wound nurse.

Warning

The ProtectorMax should not be adjusted too tight, but tensioned enough so a hand can still slide in between. The ProtectorMax should not come into direct contact with open wounds and should be worn on dry skin. The use of lotions or creams is not recommended and should first be discussed with your wound nurse or physician.

Maintenance

In order to increase the lifespan, it is advisable to store the ProtectorMax in the supplied packaging when it is not being used. This protects the ProtectorMax from deformation, external pressure effects or possible weathering influences.

Applied materials

ABS, TPU, nylon, polyamide, elastane and viscose. Latex free.

Washing instructions

We recommend cleaning the plastic cap of the ProtectorMax with a cloth and lukewarm water. The plastic cap is not suitable for washing machine or dryer. We recommend washing the elastic strap in a washing bag in warm water at 40°C / 104°F with a mild detergent.

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