Stomacare

HydroActive

Model 365

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EN Instructions for use

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Basko
Healthcare

NL - Gebruiksaanwijzing.....	4
DE - Gebrauchsanleitung	6
EN - Instructions for use.....	8
FR - Mode d'emploi	10
IT - Manuale di istruzione	12
ES - Manual de instrucciones.....	14

NL

DE

EN

FR

IT

ES

HydroActive Stomacare bandage

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HydroActive Stomacare Ostomy Belt

Instructions for use

Stomacare HydroActive belts are non-prescription products that are ordered directly from the health care supply store when needed or through your ostomy therapist. Medicinal creams, etc. may only be used under the supervision of a doctor.

Intended purpose/Indications

Stomacare HydroActive belts are primarily used as post-operative supports for the abdominal wall during physical activities to help prevent ruptures (parastomal hernia) or to give additional support in case of an existing rupture.

The Stomacare "HydroActive belt is made of a breathable neoprene material and is specially developed for use in water (showers, sauna, swimming, water aerobics, water therapy, etc.) With the application, the stoma is securely fixed, well covered and protected against excessive moisture penetration. Thus, during the activities in the water, stoma patients are offered optimal protection and a sense of security. Use by several patients is not provided.

Permissible operating conditions/Deployment

The Stomacare HydroActive belt can be used both in fresh and saltwater.

Existing health risks and contraindications

There are no health risks or other contraindications known.

Adjustment

The Stomacare HydroActive belt may only be adapted / changed by a medical professional with sufficient experience.

Package Contents

- Belt
- Instructions for use

Specification of the materials used

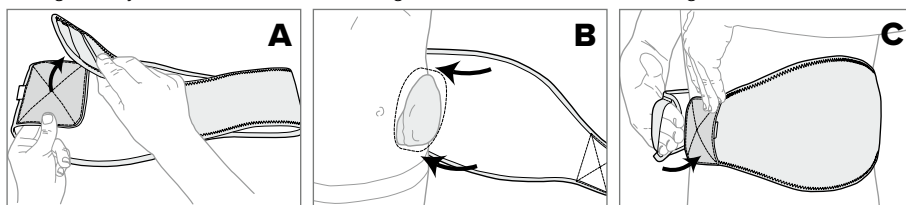
Stomacare HydroActive belts are completely latex-free, belt: 2-layer 100 % polyamide with breathable neoprene in between.

Maintenance

Stomacare HydroActive belts are low-maintenance and easy-care. To ensure a long lifetime it is recommended to wash the belt from time to time. For optimal adhesion of the closure it is recommended to keep the Velcro free of dust, lint, fibres or other impurities. Even when the belt is not being worn, it is important to store it with the closure closed.

Putting on the belt

1. Fig. **A**: Undo the Velcro closure on the belt.
2. Fig. **B**: The stoma pouch is now completely placed in the belt.
Make sure that bowel movements are not blocked.
3. Fig. **C**: Adjust the belt to the desired tightness and close the belt using the Velcro closure.

Instructions for use for patients

Stomacare HydroActive belts are used to support the abdominal wall, to prevent hernias and to prevent prolapses. Hernias in particular are caused by physical activity (e.g. sport, gardening/housework, lifting, coughing, sneezing, etc.). Wearing a Stomacare HydroActive belts during water sports can protect from these unpleasant side effects. Please check when fitting the belt whether the size provided corresponds to your current abdominal circumference (see size chart).

The Stomacare HydroActive belt is best fitted and adjusted when lying down. When adjusting the belt, it should always be possible to slide a hand between the belt and the body. This means it is tight enough and comfortable to wear. Do not tighten the belt too much. Check the condition of your skin on a daily base. Hypersensitivity to the material has so far occurred very rarely. However, should the first signs of skin irritation, redness or swelling appear, immediately remove the belt.

Washing instructions

Stomacare HydroActive belts are very durable and of high quality. We recommend washing the belt in a washing net at max. 40° (100°F) with a gentle detergent. It is not recommended to use washing powder. The product should in no case be dried in a dryer or dry cleaned. It is best to dry it flat, do not iron or bleach it. To avoid damaging the material, all Velcro closures should be closed before washing.

Size chart

Size	Abdominal circ.
Small	70 – 80 cm
Medium	80 – 90 cm
Large	90 – 100 cm
X-Large	100 – 110 cm
XX-Large	110 – 120 cm
XXX-Large	120 – 130 cm
XXXX-Large	130 – 140 cm

The size chart can be used as a guide for selecting the correct size. To determine the right size the abdominal circumference has to be measured.

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