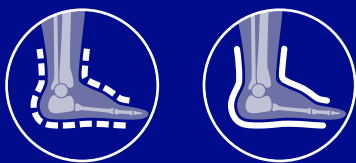
THUASNE®

# XLR8

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## INFLATABLE RIGID-SHELL WALKING BOOT

### Description/Destination

Full rigid shell walking boot to support and protect the lower third of the leg, ankle and foot.  
The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Rigid components: polyamide - rubber - ethylene vinyl acetate - aluminium - thermoplastic polyurethane - polypropylene.  
Textile components: polyamide - polyurethane - polyester.

### Properties/Mode of action

Immobilizes and stabilizes the lower third of the leg, ankle and foot.  
Two independent integrated air cells.  
Designed to ensure minimal dorsiflexion of the metatarsal bone.  
Curved, anti-slip sole that offers patients a natural, comfortable walking motion.  
Adjustable toe bar (3 possible positions).  
This is a bilateral product.

### Indications

Ankle/foot post-operative/post-traumatic immobilization.  
Severe ankle/foot sprain.  
Stable fracture of the lower third of the leg (tall version).  
Conservative treatment of ankle/foot fracture.  
Ankle/foot severe tendinopathy.  
Conservative treatment of Achilles tendon rupture.

### Contraindications

Do not use the product if the diagnosis has not been confirmed.  
Do not apply the product in direct contact with broken skin.  
Do not use in the event of known allergy to any of the components.  
Do not use in the event of unstable fractures or fractures of the upper part of the tibia or fibula.  
Do not use for patients weighing > 130 kg / 300lbs.  
Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

### Precautions

Verify the product's integrity before every use.  
Do not use the device if it is damaged.  
Choose the appropriate size to fit the patient, referring to the size chart.  
The choice of the device version is at the discretion of the healthcare professional.  
It is recommended that a healthcare professional supervises the first application.  
Strictly comply with your healthcare professional's prescription and recommendations for use.  
The systematic use of a long sock is recommended when wearing the device.  
Ensure correct position and avoid folds when placing foam liner in the device.  
Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).  
Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).  
Do not bring the product into direct contact with a dressing.  
In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.  
Do not wear the device in a medical imaging machine.  
Do not wear the device when driving a vehicle.  
Do not expose the product to extreme temperatures (> 60°C).  
It is recommended to adequately tighten the device to achieve immobilisation without restricting blood circulation. In the event of pain, tingling or numbness, loosen the straps or deflate the air pads and ask a healthcare professional for advice.  
It is recommended that the venous thromboembolic risk be assessed before any immobilisation is indicated. Follow the advice of a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.  
If the device comes into contact with water, dry the textile part and wipe the rigid part well with a dry cloth.  
Prolonged immobilisation can sometimes cause muscle weakness. When resuming walking, be careful of risks of falling.  
In case of history of venous or lymphatic disorders, consult a healthcare professional.

### Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.  
Possible risk of venous thrombosis.  
Possible risk of back pain due to gait pattern modification.  
Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

- Before putting on the walking boot, make sure the tightening system has been fully unfastened and that the liner is fully deflated.
- Remove the walking boot from its plastic cover, unfasten the hook-and-loop straps, remove the rigid front part and open the foam liner.
  - Sit down and carefully place your lower leg and foot in the foam liner, checking that your heel is correctly positioned at the bottom of the boot . Close the foam liner, applying it firmly around the lower leg and foot .
  - Reposition the rigid front part .
  - Fasten the walking boot again, tightening the self-fastening straps from bottom  to top .  
For a better tightening fit, pass the top hook-and-loop strap through the loop cut directly into the hard shell .
  - Adjust the toe protector.
  - Inflating/deflating the boot**  
To increase compression, inflate the air cells by pressing the blue bulb  1.  
To deflate the air cells, open the cap on the back of the blue bulb and press the valve to release the air .
  - Try walking with the boot to check that it fits properly and that it is working correctly.

**Taking off the walking boot:** Loosen the straps, remove the rigid front section and open the liner.

Additional accessories/spare parts are available to order.

### Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label.  
If the device comes into contact with water, dry the textile part and wipe the rigid part well with a dry cloth.  
Rigid components:  
Wash the rigid part with a moist cloth.  
Textile components:  
The soft part can be fully removed for washing.  
Replace in the original location before next use. Hand wash the liner in cold water. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

### Storage

Store at room temperature, preferably in the original packaging.

### Disposal

Dispose of in accordance with local regulations.

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© Thuasne - 6172801 (2023-09)
Thuasne UK Ltd